MASSFORCE

MASSMULATOR™ Stimulator TENS

model AS2015

modèle AS2015

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Read the manual before operating this product.

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WARNINGS

- If you are in the care of a physician, consult your physician before using this device.
- **DO NOT** use this device if you have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic devices.
- **DO NOT** use this device during pregnancy.
- **DO NOT** use this device on children.
- **DO NOT** place this device over the carotid sinus nerves, the front of the neck, or around the mouth.
- **DO NOT** open unit. The opening will void the warranty.
- **DO NOT** use this device over, or in proximity to, cancerous lesions.
- **DO NOT** place the device over open wounds, sores, or rashes, or over swollen, red, infected, or inflamed areas or skin eruptions (e.g., phlebitis, thrombosis, varicose veins).
- **DO NOT** use this device when in a bath or shower.
- **DO NOT** use this device while sleeping.
- **DO NOT** use this device while driving or operating machinery.
- **DO NOT** use this device in the presence of electromagnetic fields.
- **DO NOT** use this device if the unit has been damaged.
- **DO NOT** use on children or pets.
- **DO NOT** use in an area where water or moisture can get on the unit, such as a bathroom or other places with high humidity.

PRECAUTIONS

- The long-term effects of electrical stimulation are unknown.
- Since the effects of stimulation of the brain are unknown, stimulation should not be applied across your head.
- The safety of electrical stimulation during pregnancy has not been established.
- Use caution if you have a tendency to bleed internally, such as following an injury or fracture.
- You may experience skin irritation or hypersensitivity.
- If you have suspected or diagnosed heart disease, you should follow the precautions recommended by your physician.
- Keep this device out of the reach of children.
- If you have suspected or diagnosed epilepsy, you should follow the precautions recommended by your physician.
- Consult with your physician prior to using the device after a recent surgical procedure, because the stimulation may disrupt the healing process.
- Use caution if stimulation is applied over the menstruating or pregnant uterus.
- Use caution if stimulation is applied over areas of skin that lack normal sensation.
- Use this device only with the electrodes and accessories recommended by the manufacturer.
- Avoid placing the pads over metal implants.

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INTRODUCTION

Massmulator™ is a total package for muscle stress-free. Enjoy the 28 different modes and controls plus 20 level intensity. LED display and touch screen for the effortless gesture to control. 5-6 hours usage.

Electrical Muscle Stimulation (EMS): Used for temporary relief of pain associated with sore and aching muscles due to strain from exercise or normal household work activities.

Transcutaneous Electrical Nerve Stimulation (TENS): Used to stimulate healthy muscles in order to improve and facilitate; muscle performance.

Indications for Use

Use Massmulator™ to stimulate blood circulation and muscle pain relief in order to improve and facilitate muscle performance all around the body. It is also intended for temporary relief of pain associated with sore and muscle stiffness from exercise or normal work activities.

ADVERSE EFFECTS

- Skin irritation and burns beneath the stimulation electrodes applied to your skin can occur.
- On very rare occasions, first-time users of EMS have reported feeling light-headed or faint. We recommend that you use the product while seated until you become accustomed to the sensation.

You should stop using the device and consult with your physician if you experience adverse reactions from the device.

HOW TO USE

1. Make sure your skin clean and dry before sticking the reusable adhesive pads. Wipe with dry tissue or towel.
2. Connect the wire to the reusable adhesive pads.
3. Peel off the plastic sheet over the pads.
4. Place pads onto your targeted skin.
5. Insert two ports into the device.
6. Switch ON (on the right side).
7. Select your favorite modes and intensity.

Regular Application Principles :

1. Find the exact pain point; this is the position for the pads to be placed. The acupuncture chart can also serve as a guide for pad placement specific to each ailment.
2. Adjust the intensity gradually. It is more effective to reach the highest intensity level you can stand.
3. Recommended application duration is 2-3 times per day for 40-60 minutes per application, over 10 days for one course.

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UNIT DIAGRAM

1. Time setting : touch the time icon to adjust time to 10,20,30,40,50,60,70,80 minutes.
2. Output A or B shift button: touch icon A to shift from A to B or touch icon B to shift from B to A.
3. Safety lock icon: Press the key button once to lock the screen to disable the touch screen function; press again to unlock the screen to enable the touch screen function. When it is locked, the icon timer blinks.
4. Output A.
5. USB port for charging the battery.
6. Output B.
7. ON/OFF button: Hold the ON/OFF button for 6 seconds to run the device ON/OFF.
8. Battery level indicator.
9. Intensity increase button: 10 bar/20 levels of intensity adjustable. Press the intensity + button once to increase 1 level. Press 1-20 times to increase intensity up to the highest level of 20.
10. Intensity decrease button: 10 bar/20 levels of intensity adjustable. Press intensity - button once to decrease 1 level. Press 1-20 times to decrease intensity up to the lowest level 0.
11. Mode setting: Any icon of 12 therapeutic modes can be chosen on the touch screen.
12. Time setting button.
13. Touchscreen pen: pen to operate the screen. That is an additional accessory even you can select the screen by fingers.

Included in the box :

- Control unit
- 2 set of small single-sided adhesive electrode pads
- 2 set large single-sided adhesive electrode pads
- 2 electrode wire with 2 leads
- 1 instruction manual
- USB extension wire
- Application acupuncture points chart
- Pad holder

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MODES & FUNCTION

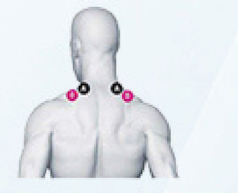
1. **Tuina:** Stimulating tuina is a hands-on body treatment that opens the flow of Qi through the meridians.
2. **Acupuncture:** Stimulating acupuncture provides the same sensation without needles, by utilizing electronic pulses to achieve the sensation of differing needle depths.
3. **Tapping:** Stimulating tapping is intended to produce a hard thumping sensation that massages your muscles, that can give you an invigorated and revitalized feeling.
4. **Gausha:** Stimulating gausha with a low-frequency setting provides the user with scratching and a kneading sensation. Gausha releases unhealthy elements from injured areas and stimulates blood flow and healing.
5. **Cupping:** Stimulating cupping, relatively constant stimulation contracts muscles.
6. **Combination:** Auto-combination of several modes for relaxation.
7. **Shiatus:** Stimulating Shiatus is a type of alternative medicine consisting of finger and palm pressure, stretches, and other massage techniques.
8. **Deep Tissue Massage:** Deep tissue massage therapy is similar to Swedish massage, but the deeper pressure is beneficial in releasing chronic muscle tension.
9. **Back pain:** Use low-frequency vibrations to promote circulation of blood and relieve pain.
10. **Pressure:** It is the manipulation of superficial and deeper layers of muscle and connective tissue using various techniques.
11. **Body Building:** Applying electrical current to muscles causes muscles to contract. Stimulating muscles repeatedly with electricity may eventually result in muscles that are strengthened and toned to some extent.
12. **Weight Loss:** Combination of acupuncture and other waveforms.

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

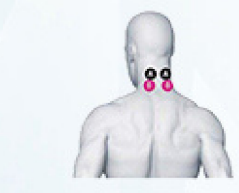
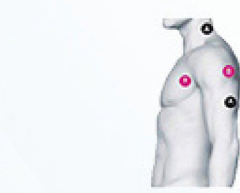
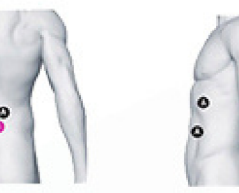
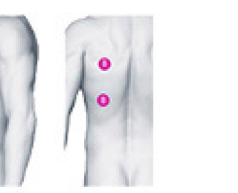
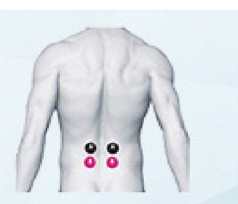
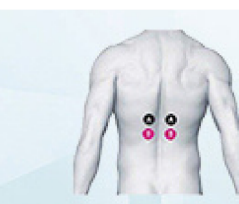
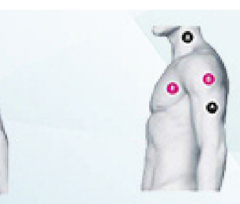
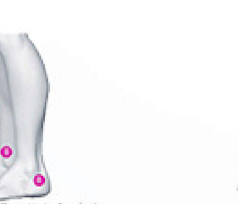
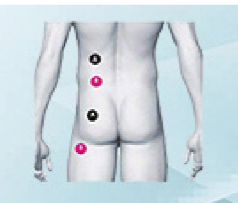
Hip Neuralgia
Névralgie de la hanche

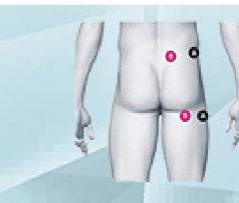
Sciatica
Sciatique

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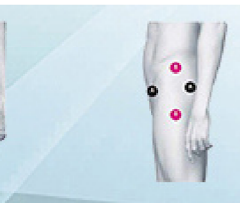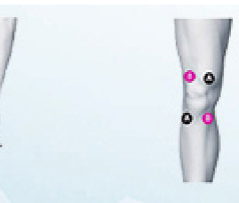
Carpal Tunnel Syndrome
Syndrome du canal carpien

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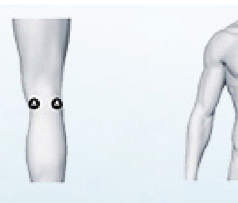MAINTENANCE

1. Make sure your skin is free from any dirt, oil, or lotions.
2. Before applying the pads, put few drops of water on your fingers and rub them on both pads. This will help the pads maintain their adhesiveness. When you have finished using the unit, turn it off.
3. Take off the pads.
4. Place the protective film back on the pads or place the pads on the pad holder.
5. Take the wire out from the device and place it back in the box along with the unit for next use.
6. Store the unit in a cool, dry place.

TROUBLESHOOTING

- **Stimulation is weak or non-existent** : Be sure skin is clean and pads are firmly attached.
- **Device does not turn on** : Recharge for 20 minutes.
- **Gel pads are not sticky** : The gel pads will lose adhesiveness over the course of use. Please visit [Massforce.co](https://massforce.co) for replacements.

Massforce branded items purchased from massforce.co include a 1-year limited replacement warranty. For Customer Service, please send us email: hello@massforce.co.

DO NOT DISPOSE OF BATTERIES IN FIRE. ALWAYS DISPOSE OF BATTERIES PER LOCAL & FEDERAL GUIDELINES. The SHARPER IMAGE® name and logo are registered trademarks.

©Massforce. All Rights Reserved.

Read Instructions
Before Operating

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